



ADDITIONAL INFORMATION

Weight	0.000998 kg
Dimensions	12 × 3.5 × 7.5 cm
Carton Qty	48
Pricing Unit	Box/20
Size	4cm x 5cm
Min Order Qty	1
ARTG ID	159198
GMDN Code	44990
Medical Device Class	Class 1s
UNSPSC	42311527
Tax Schedule	Non-Taxable
Item Length	5
Item Width	4
Item Measurement Unit	cm
Is Serialized Item	No
Is Dangerous Good	No
Shelf Life (Months)	36

PRODUCT DESCRIPTION

This post-operative dressing combines the advantages of AEROFILM with a SupaSorb™ absorbent and low adherent dressing pad to provide a superior dressing for post operative and delicate wound sites.



AEROFILM PLUS WATERPROOF ISLAND DRESSING

4 X 5CM BOX/20 BOX / 20

AFPW00420

Ultra-thin, Waterproof & Transparent Bacterial Barrier

Supabond™

Hypo-allergenic adhesive
exclusively engineered,
for low irritation and high adhesion

SuperSorb™ Wound

Exudate Pad
Super Absorption,
Reducing Skin Maceration

Non-stick Barrier

Protects Wound with a Non-Adherent Barrier,
Minimising Trauma on Removal

Dressing Frame

Ensures Easy, Effective Application

SupaBond™ Hypo-allergenic Adhesive

- Exclusively engineered for unrivalled adhesion, this hypoallergenic adhesive has a low irritation risk while ensuring the dressing stays in place over body contours.

Transparent AeroFilm

- The transparent AeroFilm allows the user to monitor the level of fluid and check for clinical signs of infection without removing the dressing. This allows for maximum usage of the dressing before removal.

Simple Application

- AeroFilm has an integrated frame applicator that provides a simple and effective application system. The frame enables the dressing to be applied crease free and ensures adhesion. AeroFilm transparency enables correct positioning over the wound site.



AEROFILM PLUS WATERPROOF ISLAND DRESSING 4 X 5CM BOX/20 BOX / 20

AFPW00420

Contraindications / Precautions

- AeroFilm should not be used on infected wounds unless under the supervision of a healthcare practitioner.
- As with all adhesive products, apply and remove carefully from sensitive or fragile skin.
- Seek medical advice immediately for all serious wounds and burns. If redness or discomfort occurs, discontinue use and consult a physician.

This product is not a substitute for direct medical attention from a healthcare practitioner.

CONTENTS LISTS

Code	Description	Qty
AFPW004	AEROFILM PLUS Waterproof Island Dressing 4 x 5cm	1

FILM DRESSING PAD