

AEROFILM PLUS WATERPROOF ISLAND DRESSING 6 X 7CM BOX/3

AFPW0063





ADDITIONAL INFORMATION

Weight 0.023 kg

Dimensions $1.5 \times 9.5 \times 15 \text{ cm}$

Carton Qty 96

Pricing Unit Box/3

Size 6cm x 7cm

Min Order Qty 1

ARTG ID 159198 **GMDN Code** 44990

Medical Device Class 1s

UNSPSC 42311527

Tax Schedule Non-Taxable

Item Length7Item Width6

Item Measurement Unit cm

Is Medical Device Yes
Is Serialized Item No

Is Dangerous Good No

36

Shelf Life (Months)

PRODUCT DESCRIPTION

This post-operative dressing combines the advantages of AEROFILM with a SupaSorb^{TM} absorbent and low adherent dressing pad to provide a superior dressing for post operative and delicate wound sites.



AEROFILM PLUS WATERPROOF ISLAND DRESSING 6 X 7CM BOX/3

AFPW0063

AeroFilm Ultra-thin, Waterproof & Transparent Bacterial Barrier

Supabond™ Hypo-allergenic adhesive exclusively engineered, for low irritation and high adhesion

SuperSorb™ Wound Exudate Pad Super Absorption, Reducing Skin Maceration

Non-stick Barrier Protects Wound with a Non-Adherent Barrier, Minimising Trauma on Removal

Dressing Frame

Ensures Easy, Effective Application

SupaBond™ Hypo-allergenic Adhesive

• Exclusively engineered for unrivalled adhesion, this hypoallergenic adhesive has a low irritation risk while ensuring the dressing stays in place over body contours.

Transparent AeroFilm

• The transparent AeroFilm allows the user to monitor the level of fluid and check for clinical signs of infection without removing the dressing. This allows for maximum usage of the dressing before removal.

Simple Application

AeroFilm has an integrated frame applicator that provides a simple and effective application system. The frame enables the
dressing to be applied crease free and ensures adhesion. AeroFilm transparency enables correct positioning over the wound
site.

Contraindications / Precautions



AEROFILM PLUS WATERPROOF ISLAND DRESSING 6 X 7CM BOX/3

AFPW0063

- AeroFilm should not be used on infected wounds unless under the supervision of a healthcare practitioner.
- As with all adhesive products, apply and remove carefully from sensitive or fragile skin.
- Seek medical advice immediately for all serious wounds and burns. If redness or discomfort occurs, discontinue use and consult a physician.

This product is not a substitute for direct medical attention from a healthcare practitioner.

This product is not a substitute for direct medical attention from a healthcare practitioner. Always read the label and follow the directions for use.

Film Dressing Pad