

AEROFILM WATERPROOF FILM DRESSING 10 X 12CM BOX/20

AFW010





ADDITIONAL INFORMATION

Weight 0.14 kg

Dimensions $19.7 \times 3.5 \times 14 \text{ cm}$

Carton Qty 8

Pricing Unit Box/20

Size 10cm x 12cm

Min Order Qty 1

ARTG ID 159198 **GMDN Code** 44990

Medical Device Class 1s

UNSPSC 42311527

Tax Schedule Non-Taxable

Item Length12Item Width10

Item Measurement Unit cm

Is Medical Device Yes
Is Serialized Item No

Is Dangerous Good No

Shelf Life (Months) 36

PRODUCT DESCRIPTION

AeroFilm is a sterile, waterproof, ultra-thin, transparent film dressing. It is designed to be applied over I.V. devices, clean skin or wounds and provides a breathable anti-bacterial and viral barrier to outside contaminants.



AEROFILM WATERPROOF FILM DRESSING 10 X 12CM BOX/20

AFW010

AeroFilm Ultra-thin, Waterproof & Transparent Bacterial Barrier

Supabond™ Hypo-allergenic adhesive exclusively engineered, for low irritation and high adhesion

Dressing Frame Ensures Easy, Effective Application

SupaBond adhesive

• Exclusively engineered for unrivalled adhesion, ensures that AeroFilm can be left in place for up to 7 days. This reduces interference with the wound, reducing nursing time and aiding faster recovery.

Waterproof Film

• AeroFilm is impermeable to water and body fluids, protecting the application underneath and allowing the patient to shower and bathe without redressing. It also aids in the prevention of bacterial contamination and reduces risks of skin damage through interference.

Simple application

AeroFilm has an integrated frame applicator that provides a simple and effective application system. The frame enables the
dressing to be applied crease free and ensures adhesion. AeroFilm transparency enables correct positioning over the wound
site

Adaptable to Awkward Areas

• AeroFilm can be easily applied to the most awkward body contours, due to its expandability, while SupaBond ensures the dressing remains in place.

Contraindications / Precautions

- AEROFILM should not be used on infected wounds unless under the supervision of a healthcare practitioner.
- As with all adhesive products, apply and remove carefully from sensitive or fragile skin.
- Seek medical advice immediately for all serious wounds and burns. If redness or discomfort occurs, discontinue use and consult a physician.

This product is not a substitute for direct medical attention from a healthcare practitioner.

This product is not a substitute for direct medical attention from a healthcare practitioner. Always read the label and follow the directions for use.

Waterproof Film Dressing



AEROFILM WATERPROOF FILM DRESSING 10 X 12CM BOX/20

AFW010