

ADDITIONAL INFORMATION

Weight	0.24 kg
Dimensions	30 × 3.5 × 18.5 cm
Carton Qty	20
Pricing Unit	Box/20
Size	15cm x 20cm
Min Order Qty	1
ARTG ID	159198
GMDN Code	44990
Medical Device Class	Class 1s
UNSPSC	42311527
Tax Schedule	Non-Taxable
Item Length	20
Item Width	15
Item Measurement Unit	cm
Is Serialized Item	No
Is Dangerous Good	No
Barcode (GTIN/EAN/UPC)	9341394002364



PRODUCT DESCRIPTION

AeroFilm is a sterile, waterproof, ultra-thin, transparent film dressing. It is designed to be applied over I.V. devices, clean skin or wounds and provides a breathable anti-bacterial and viral barrier to outside contaminants.

Ultra-thin, Waterproof & Transparent Bacterial Barrier

Supabond™
Hypo-allergenic adhesive
exclusively engineered,
for low irritation and high adhesion

Dressing Frame
Ensures Easy, Effective Application

SupaBond adhesive

- Exclusively engineered for unrivalled adhesion, ensures that AeroFilm can be left in place for up to 7 days. This reduces interference with the wound, reducing nursing time and aiding faster recovery.

Waterproof Film

- AeroFilm is impermeable to water and body fluids, protecting the application underneath and allowing the patient to shower and bathe without redressing. It also aids in the prevention of bacterial contamination, and reduces risks of skin damage through interference.

Simple application

- AeroFilm has an integrated frame applicator that provides a simple and effective application system. The frame enables the dressing to be applied crease free and ensures adhesion. AeroFilm transparency enables correct positioning over the wound site.

Adaptable to Awkward Areas

- AeroFilm can be easily applied to the most awkward body contours, due to its expandability, while SupaBond ensures the dressing remains in place.



AEROFILM WATERPROOF FILM DRESSING 15 X 20CM BOX/20

AFW015

Contraindications / Precautions

- AEROFILM should not be used on infected wounds unless under the supervision of a healthcare practitioner.
- As with all adhesive products, apply and remove carefully from sensitive or fragile skin.
- Seek medical advice immediately for all serious wounds and burns. If redness or discomfort occurs, discontinue use and consult a physician.

This product is not a substitute for direct medical attention from a healthcare practitioner.

Waterproof Film Dressing