

AEROPLAST WOUND CLOSURE STRIPS 3 X 75MM 5 STRIPS/CARD BOX/50

APWC7535





ADDITIONAL INFORMATION

Weight 0.06 kg

Dimensions $2.5 \times 6.5 \times 13.5 \text{ cm}$

Carton Qty 100

Pricing Unit Box/50

Size 3mm x 75mm

Min Order Qty 1

ARTG ID 159198 **GMDN Code** 44990

Medical Device Class 1s

UNSPSC 42311546

Tax Schedule Taxable

Item Length7.5Item Width0.3

Item Width 0.3
Item Measurement Unit cm

Is Medical Device Yes

Is Serialized Item No

Is Dangerous Good No

Shelf Life (Months) 36

PRODUCT DESCRIPTION

The AEROPLAST Wound Closure strips offer you a painless alternative for the closure of minor wounds without the need for stitching.

There is nothing to show here! Slider with alias aeroplast-wound-closures-layers not found.

SupaBond™ Adhesive

• SupaBond adhesive, exclusively engineered for unrivalled adhesion, provides a secure adhesion across the site, preventing lift



AEROPLAST WOUND CLOSURE STRIPS 3 X 75MM 5 STRIPS/CARD BOX/50

APWC7535

up or edge curling.

Comprehensive sizes available

• AeroPlast wound closures are available in sizes from 38 to 100mm long and from widths of 3mm to 100mm, enough to suit any incision or laceration. They can also be cut to size.

Neat and Simple

• AeroPlast Wound closures offer fast and easy application with no need for an anaesthetic. AeroPlast wound closures reduce scarring and eliminate needle puncture marks.

Contraindications / Precautions

- Aeroplast should not be used on infected wounds unless under the supervision of a healthcare practitioner.
- As with all adhesive products, apply and remove carefully from sensitive or fragile skin.
- Seek medical advice immediately for all serious wounds and burns. If redness or discomfort occurs, discontinue use and consult a physician.

This product is not a substitute for direct medical attention from a healthcare practitioner.

This product is not a substitute for direct medical attention from a healthcare practitioner. Always read the label and follow the directions for use.

Wound Closure Strips - cards of 5