

## AEROSTEP FOAM CORN CUSHIONS 20 X 26MM

ASTCC09



## **ADDITIONAL INFORMATION**

Weight 0.01 kg

**Dimensions**  $14.7 \times 7.6 \times 2 \text{ cm}$ 

Carton Qty 585

Pricing Unit Box/9

Min Order Qty 1

**ARTG ID** 428730 **GMDN Code** 61945

Medical Device Class 1

Tax Schedule Taxable

Is Medical Device Yes

Is Serialized Item No

Is Dangerous Good No

Shelf Life (Months) 60

## PRODUCT DESCRIPTION

AeroStep™ Corn Cushions help relieve painful shoe pressure and discomfort caused by rubbing. The soft cushioning foam shape allows socks and tights to glide over the cushion.

**DIRECTIONS:** Clean and dry area before use. Remove film backing. Apply the cushion on the affected area with the corn in the centre hole. Replace at least every 16 hours.

SINGLE USE ONLY. TOPICAL USE ONLY - DO NOT PUT IN CONTACT WITH OPEN WOUND. DO NOT CLEAN WITH ALCOHOL. KEEP OUT OF REACH OF CHILDREN. IF THE PRODUCT CAUSES ANY DISCOMFORT, REDNESS OF SKIN OR EXACERBATES PAIN, DISCONTINUE USE AND SEEK PROFESSIONAL ADVICE. THIS PRODUCT IS NOT A SUBSTITUTE FOR PROFESSIONAL ATTENTION FROM A HEALTHCARE PRACTITIONER.



directions for use.

## AEROSTEP FOAM CORN CUSHIONS 20 X 26MM

ASTCC09

This product is not a substitute for direct medical attention from a healthcare practitioner. Always read the label and follow the

DIABETICS AND PEOPLE WITH BLOOD CIRCULATION PROBLEMS SHOULD CONSULT WITH A PHYSICIAN BEFORE USE.