



ADDITIONAL INFORMATION

Weight	0.023 kg
Dimensions	14.7 × 7.6 × 2 cm
Carton Qty	585
Pricing Unit	Box
Min Order Qty	1
ARTG ID	426930
GMDN Code	36295
Medical Device Class	Class 1
Tax Schedule	Taxable
Is Serialized Item	No
Is Dangerous Good	No
Shelf Life (Months)	60

PRODUCT DESCRIPTION

AeroStep™ Gel Sore Spots are made from soft gel with a self-adhesive backing. It provides pressure and friction relief and is difficult to see when in the shoe.

DIRECTIONS: Remove film backing. Apply adhesive side where required. Wash with warm water and let dry. Use for more than 16 hours per day is not recommended.

TOPICAL USE ONLY - DO NOT PUT IN CONTACT WITH OPEN WOUNDS. DO NOT CLEAN WITH ALCOHOL. KEEP OUT OF REACH OF CHILDREN.

THIS PRODUCT IS NOT A SUBSTITUTE FOR PROFESSIONAL ATTENTION FROM A HEALTHCARE PRACTITIONER.

DIABETICS AND PEOPLE WITH BLOOD CIRCULATION PROBLEMS SHOULD CONSULT WITH A PHYSICIAN BEFORE USE.



AEROSTEP GEL SORE SPOTS 6PK

ASTSS06