



ADDITIONAL INFORMATION

Weight	0.025 kg
Dimensions	14.7 × 7.6 × 0.2 cm
Carton Qty	585
Pricing Unit	Box
Min Order Qty	1
ARTG ID	426930
GMDN Code	36295
Medical Device Class	Class 1
Tax Schedule	Taxable
Is Medical Device	Yes
Is Serialized Item	No
Is Dangerous Good	No
Shelf Life (Months)	60



PRODUCT DESCRIPTION

AeroStep™ Gel Toe Spreader separates the first and second toes, holding overlapping toes in their correct position. Made of an elastic, soft, moisturising gel, it aids in the prevention of bunions.

DIRECTIONS: Place between the toes. Remove overnight. Wash with warm water and let dry. Use for more than 16 hours per day is not recommended.

TOPICAL USE ONLY - DO NOT PUT IN CONTACT WITH OPEN WOUNDS. DO NOT CLEAN WITH ALCOHOL. KEEP OUT OF REACH OF CHILDREN.

THIS PRODUCT IS NOT A SUBSTITUTE FOR PROFESSIONAL ATTENTION FROM A HEALTHCARE PRACTITIONER.

DIABETICS AND PEOPLE WITH BLOOD CIRCULATION PROBLEMS SHOULD CONSULT WITH A PHYSICIAN BEFORE USE.



AEROSTEP GEL TOE SPREADER AND BUNION PROTECTOR 1PK

ASTBP01

This product is not a substitute for direct medical attention from a healthcare practitioner. Always read the label and follow the directions for use.