



ADDITIONAL INFORMATION

Weight	1 kg
Carton Qty	1
Pricing Unit	Unit/1
Size	Adult
Min Order Qty	1
UNSPSC Code	42301503
Tax Schedule	Taxable
Is Serialized Item	No
Is Dangerous Good	No



PRODUCT DESCRIPTION

HOW TO CONVERT AN ADULT TO A JUNIOR:

1. Open up manikin and remove adult spring.
2. Insert spring adaptor into the base of the manikin.
3. Connect junior spring into the spring adaptor.
4. Connect the second spring adaptor to the rib frame.
5. Close the rib section, ensuring prongs are placed into the slots.
6. Replace adult face and body skin with junior face and body skin.

Compatible with:

[Brayden](#), [Brayden Pulse](#) & [Brayden Pro](#)

This product is not a substitute for direct medical attention from a healthcare practitioner. Always read the label and follow the directions for use.

Brayden CPR Manikin (no illuminators)