





ADDITIONAL INFORMATION

Weight 1 kg
Carton Qty 1

Pricing Unit Unit/1
Size Adult

Min Order Qty 1

UNSPSC 42301503
Tax Schedule Taxable
Is Serialized Item No

Is Dangerous Good No

PRODUCT DESCRIPTION

HOW TO CONVERT AN ADULT TO A JUNIOR:

- 1. Open up manikin and remove adult spring.
- 2. Insert spring adaptor into the base of the manikin.
- 3. Connect junior spring into the spring adaptor.
- 4. Connect the second spring adaptor to the rib frame.
- 5. Close the rib section, ensuring prongs are placed into the slots.
- 6. Replace adult face and body skin with junior face and body skin.



BRAYDEN JUNIOR CPR MANIKIN KIT

IM13-J

Compatible with:

Brayden, Brayden Pulse & Brayden Pro

This product is not a substitute for direct medical attention from a healthcare practitioner. Always read the label and follow the directions for use.

Brayden CPR Manikin (no illuminators)